




- **All STELLEST™ lens orders must be ordered as Complete or Store Sends Frame**
- **Reference processing guidelines for all Rx ranges**
- **Reference table below for special order lab flow**
- **For Store Sends Frame, reference Order Tracker for Central Manufacturing location**


Special Processing Type	How to Order in Lab Processing	Applicable For
Outside Processing – Central Manufacturing Provides	No processing change required; transmit order as Complete – Central Manufacturing Provides Frame	Complete Pair Sales & Complete Pair Exchanges
Outside Processing – Store Sends Frame	No processing change required; transmits as Store Sends Frame	Frame unavailable through Central Manufacturing, Customer Own Frame, Lens Only exchange, Stellest Lens Warranty exchange
Outside Processing – Lens Only – Uncut or Cut & Edge	You MUST change the processing for this order to Store Sends Frame	All sales/exchange types

STELLEST™ lenses will always arrive from Central Manufacturing as a complete pair, never lenses only. Use this guide as reference for final inspection.

Start by locating the center lenslet from the back of the lens. Use the overhead light and black background to create contrast as you angle the frame until you can clearly see the center ring and then dot along the edge.

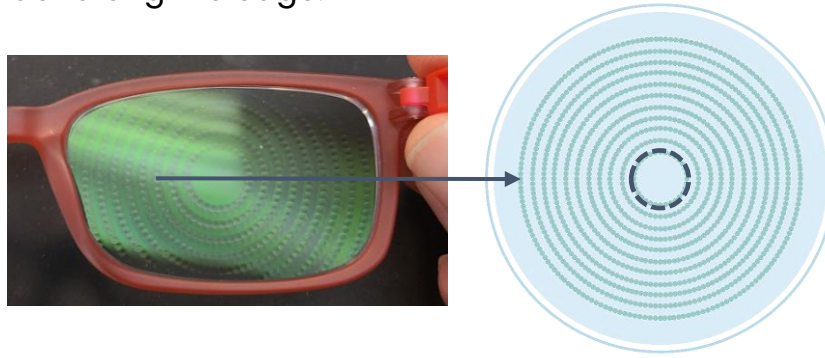

Once the center ring is dotted utilize Eyewear Inspection Job Aid to **mark the center dot by using the HD SV 0 mm seg drop**. The dotted ring is just slightly larger than the circle on the job aid so it will line up just on the inside. This center reference point is what you will use to center in the lensometer reticle for inspection.

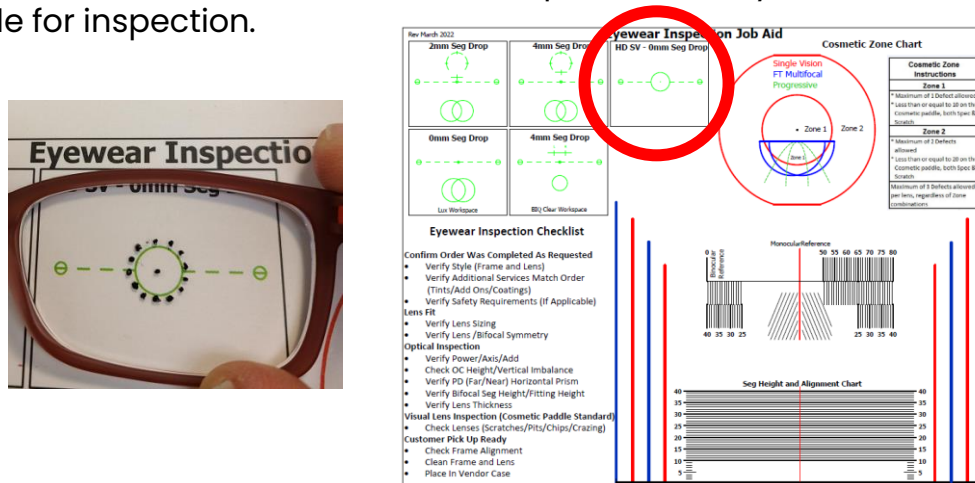

Place the lens in the lensometer using the center dot as reference.



Before clamping verify with inker pen that the lens is truly centered.



Once verified clamp the lens and continue inspection checking:

- ✓ Power
- ✓ Axis
- ✓ Prism
- ✓ PD
- ✓ OC